

# Carbon Ultralight



Carbon Ultralight should be regularly maintained by removing dust and dirt. Pay special attention to keeping the brakes clean to ensure that they always work as intended. All surfaces can be cleaned with a textile cloth and clean water, if necessary with a little non-abrasive detergent. Wipe dry afterwards. Do not use paper towels as these in some cases can scratch the surfaces.

Occasionally it may be necessary to tighten one or more screws. This is done with a 3mm Allen wrench. Parts subject to wear such as wheels and brakes should be replaced in good time before it can pose a security risk. Any defective part must be replaced immediately. Never use the Carbon Ultralight if it is in any way defective. Contact your dealer for servicing. Always check that the brakes work properly before any use of the Carbon Ultralight. They are important for your safety. One of the advantages of a very lightweight rollator is that it

becomes easier to go up and down slopes. You do not have to push as much weight uphill and there is not so much weight that can run from you downhill. You must always be aware that the stability of Carbon Ultralight changes when you walk on slopes. Pay special attention if you walk on a sideward slope.

Although Carbon Ultralight features large flexible wheels that make it easy and comfortable to walk on uneven surfaces, it is important that you look carefully where you go.

The seat of Carbon Ultralight should only be used when both parking brakes are activated. It is not suitable for transporting other persons. Carbon Ultralight is not intended for the transport of children. A rollator is in no way a toy for children. Keep an eye on children to ensure they do not climb on the rollator.

# Product Specifications

Product dimensions:

Technical Data	Regular	Wide Track
Rollator weight	4,80 kg	4,95 kg
Handle height (m.m.)	790-910	790-910
Grip to brake distance	62 mm	62 mm
Width between handles	390 mm	440 mm
Seat width	420 mm	470 mm
Seat height	610 mm	610 mm
User weight	Max 130 kg	Max 130 kg
Wheel width	37 mm	37 mm
Wheel diameter	200 mm	200 mm
Turning radius	750 mm	800 mm
Width	625 mm	665 mm
Width when folded	250 mm	250 mm
Length	660 mm	660 mm
Folded measures (L x W x H) in mm	660 x 250 x 800	660 x 250 x 830
Transportation weight	6,5 kg	6,7 kg

## Box dimensions

Width:	250 mm	250 mm
Height:	800 mm	830 mm
Length:	750 mm	750 mm

## Colour & EAN/SKU

Oyster White	5713504000456	5713504000227
Strawberry Red	5713504000463	5713504000234
Carbon Black	5713504000449	5713504000135

## Material

**Frame:** 100% carbon composite.

**Other parts:** Aluminum & plastic.

